



- Remove cover by placing finger under the device to stabilize
- Grasp tab on cover with other hand



- Lift tab to completely detach cover from anchor base



- Remove catheter - Do not use excessive force
- Hold pressure at site to achieve hemostasis



- Use a blunt tip scissors (Metzenbaum, Mayo or similar) to cut anchor base completely in half lengthwise along the groove



- Anchor base must be in two separate pieces



- Place one hand near the insertion site to stabilize the tissue
- Use a swift, deliberate tug to remove each half of the anchor base separately
- The flexible anchor will straighten as it is pulled out and will not cause tearing or trauma to the tissue

**24-Hour Clinical Information Line 800-225-0000**

**Go to [www.securacath.com](http://www.securacath.com) for more information**

*Please refer to instructions for use for indications, contraindications, hazards, warnings, cautions and directions for use.*

CE 0473